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Peak VN Videolaryngoskop

Hersteller:

Peak Medical Ltd.

10 Giborey Israel Str. P.O.B.
4044 Netanya 4210401, Israel

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KREIENBAUM Neoscience GmbH

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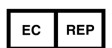
Obelis s.a.
Belgien
mail@obelis.net

Peak VN Neonatal Video Laryngoscope kitPRODUCT DESCRIPTION

The Peak VN monitor MO-2, in conjunction with reusable VNScope handles and single-use video laryngoscopes blades VNBlade, is designed to deliver clear airway views and enable rapid intubation. Low-profile designs and innovative construction make these blades streamlined and lightweight, offering improved maneuverability and working space for routine and difficult airways. With more scope options, including NOX insufflation attachment and NOX-S insufflation attachment with guiding tube, clinicians can choose their preferred tools for a wide range of patients and clinical settings.

INTENDED USE

The monitor with cameras/handles and blades is intended to work for laryngoscopy within the airway of neonates from preemies with very low birth weight to newborns with normal weight. The blades are of 4 sizes MILLER type from as small as 000 to 1. The shape blades and dimensions of the blades comply with Common Specification of such type of the blades. There for the intubations could be performed even with no image on the monitor.

Peak VN kit PARTS & ACCESSORIES

#	Catalog reference number	Qty units	Commercial name	Generic Device Term	Description	GMDN code	Remark
<u>1</u>	VNS 1	1	VNScope 1	Laryngoscope handle	Laryngoscope handle for endotracheal intubation	<u>46830</u>	
<u>2</u>	VNS 0	1	VNScope 0	Laryngoscope handle	Laryngoscope handle for endotracheal intubation	<u>46830</u>	
<u>3</u>	VNS 00	1	VNScope 00	Laryngoscope handle	Laryngoscope handle for endotracheal intubation	<u>46830</u>	
<u>4</u>	VNS 000	1	VNScope 000	Laryngoscope handle	Laryngoscope handle for endotracheal intubation	<u>46830</u>	
<u>5</u>	MO-2	1	Monitor MO2	Monitor	Monitor LCD 5"	-----	
<u>6</u>	VNB 1	-	VNBlade 1	Laryngoscope blade	Laryngoscope blade for endotracheal intubation	<u>46828</u>	Not included Purchased separately
<u>7</u>	VNB 0	-	VNBlade 0	Laryngoscope blade	Laryngoscope blade for endotracheal intubation	<u>46828</u>	Not included Purchased separately
<u>8</u>	VNB 00	-	VNBlade 00	Laryngoscope blade	Laryngoscope blade for endotracheal intubation	<u>46828</u>	Not included Purchased separately
<u>9</u>	VNB 000	-	VNBlade 000	Laryngoscope blade	Laryngoscope blade for endotracheal intubation	<u>46828</u>	Not included Purchased separately
<u>10</u>	NOX	-	NOX	Attachment	Laryngoscope handle for insufflation	<u>46828</u>	Not included Purchased separately
<u>11</u>	NOX-S	-	NOX-S	Attachment	Laryngoscope handle for insufflation with guide	<u>46828</u>	Not included Purchased separately
<u>12</u>	DNS 100	1	DNScope	Laryngoscope handle	Laryngoscope handle for endotracheal intubation	<u>47806</u>	
<u>13</u>	Cab.		<u>Cable</u>	Cable	Cable USB-A to Mini USB	-----	
<u>14</u>	Adapter	-	Adapter	Adapter	Adapter for charging Input 220VAC; Output 5VDC:1.5-2.0A Output Socket USB-A Medical Grade IEC 60601-1 approved	-----	Not included Purchased separately By a user. FRIWO FOX6-XM-USB or similar

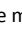
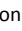
Peak VN Neonatal Video Laryngoscope kit

Instruction for Use

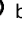
Warnings and Cautions

- It is the user's responsibility to read and to understand these instructions prior to the application of Peak Medical laryngoscopes.
- Tracheal intubation requires specialized training. Only personnel who have received training should be authorized to use video laryngoscopes.
- Don't use if the package of disposable products is opened or damaged
- Do not attempt to use any laryngoscope unless pre-use checks as described below have been carried out successfully.
- Store in dry clean condition away from heat and light.
- In the vicinity of patients and in the treatment area use the monitor only closed into the protective enclosure.
- Avoid penetration of liquids into the monitor
- Charge the monitor battery after each application, if it not in use charge at least once a week. **Don't charge the battery during intubation**
- Don't perform Charge of the battery, Set up of the monitor and File management in the vicinity of patients and in the treatment area.
- **Except battery charging and visual inspection of all components, regular technical service is not required.**

Pre-use checks and preparations prior to commencing an intubation procedure:

- Put monitor into the protective enclosure and lock it properly (latch clicked into place)
- Choose VNScope and blade size in accordance with the weight of newborn (make sure that sizes and color-coding are similar for both)
- Remove protective caps
- Check VNScope handle and blade for sharp edges or flaws that may have resulted from mechanical damage.
- Ensure that the blade is securely and correctly clicked into place on the handle.
- Connect VNScope cable securely to "Av-In" socket of the monitor (**before switch it On**)
- Switch On monitor by a short press on  button of the monitor, **wait for 10-15 sec** and check the quality of the image on the screen and battery charge level.
- Don't use the monitor if the battery is not fully charged according to the bar indicator or image is not acceptable.
- Switch Off the monitor by press on  button of the monitor for **3 secs**.
- If insufflation or guidance is required, insert the connecting pin of NOX/NOX-S in the corresponding hole of VNScope handle, direct orifice along the blade and secure it.
- Connect NOX/NOX-S with blender and adjust blender to desired oxygen concentration and flow. Check gas flow at the distant end of blade
- Open support and install the monitor steadily in the best position for intubation.
- Make sure that a view angle during intubation will be in a range of 30Deg.

Operating Instruction

- Switch On monitor by press  button of the monitor and **wait for 10-15 sec**.
- Open gas flow (if applied)
- If a recording is required, press on  button to open information screen, press on  button again to start recording, to stop recording press on  button to open information screen, press on  button again to stop recording. **SD memory card should be inserted prior** to switching On the monitor
- Intubate by applying technique recommended for video laryngoscopy
- After intubation Switch Off the monitor by press on  button of the monitor for **3 secs**.
- Discard the blade and the attachment after use and dispose them according to standard hospital procedure. Don't sterilize.
- Close the camera and the plug of the VNScope with protective caps.
- Take out the monitor from protective the enclosure for cleaning and disinfection of the enclosure, battery charge and file management.
- For management of the recorded files and set up of the monitor see **MO 2 User Guide**

Routine care and maintenance

- Video laryngoscope handles should always be cleaned and thoroughly dried before storing in a dry place.
- Keep lens always protected with cap and avoid contact with sharp and abrasive objects and materials and contamination.
- In case of contamination of the optical surface, clean it according to the procedure recommended for optical systems.
- Battery of the monitor should be recharge after each application.

Cleaning, disinfection and sterilization

- Always comply with local health&safety regulations
- Clean and disinfect handle according to standard hospital procedure for similar devices.
- When using disinfectant or sterilizing solution, always observe manufacturer's recommendation.
- Do not use chemical sterilants or disinfectants that are not approved by Peak Medical Ltd.
- Immediately after use, the VNScope should be rinsed in clean tap water to remove any residue. (**be sure that the plug and the camera are properly protected by caps**).
- External surfaces should be gently scrubbed in soapy water with a soft brush to prevent encrusted deposits. Do not use abrasive materials such as steel wool.
- VNScope may be immersed in solution (**be sure that the plug and the camera are properly protected by caps**).
- After exposure to the solution, rinse the VNScope handle with warm sterile water and dry thoroughly.
- The blades and the attachments are supplied not sterile, if required, blades could be sterilized in original packaging by EtO only. **Don't use other sterilization or disinfection procedures**
- Ultrasonic cleaning and autoclaving are not recommended and will void warranty.

Manufactured by: Peak Medical Ltd.10 Giborey Israel Str. P.O.B. 4044 Netanya 4210401, Israel, 2019

Obelis s.a.

Bd Général Wahis 53
1030 Brussels
Belgium
Tel: +(32) 2 732-59-54
Fax: +(32) 2 732-60-03
mail@obelis.net

Obelis s.a.

Bd Général Wahis 53
1030 Brussels
Belgium
Tel: +(32) 2 732-59-54
Fax: +(32) 2 732-60-03
mail@obelis.net

Manufacturer name and address Product non sterile Special operation instructions Date by which device should be used in safety Device is for single use Batch number EC n:

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User Guide

Warnings and Cautions

- It is the user's responsibility to read and to understand these instructions prior to use of Peak Medical laryngoscopes.
- Tracheal intubation requires specialized training. Only personnel who have received training should be authorized to use video laryngoscopes.
- Don't attempt to use any laryngoscope unless pre-use checks, as described in IFU of VNScope, have been carried out successfully.
- Store in dry clean condition away from heat and light.
- Charge monitor battery if it not in use at least once a week
- **Don't charge battery during intubation**
- Don't use liquid or aerosol cleaners and disinfectors other that specified by manufacturer
- Don't spill liquids of any kind onto product.
- Never push objects of any kind into this product through opening
- Don't attempt to service this product yourself. Opening or removing cover will void warranty.
- Don't insert or pull out memory TF card when the monitor is ON
- In the vicinity of patients and in the treatment area use the monitor only closed into the protective enclosure

Troubleshooting

- If no image on the screen: turn off the monitor, plug off the camera, pull out the memory card, connect the power adapter, and turn on again.
- If still no image: turn off the monitor and continue to charge for additional 2-3 hours, plug off the adapter and turn the monitor on. If the image appears, continue the charging.

Buttons and Connections

On-Screen Icons

Basic operations

1. Charging the battery

Internal Li-Ion battery could be charged via Mini USB port with supplied 5V adapter and USB cable or via 5-12V port with a special adapter (is not supplied).

2. Power ON

Plug-in connector of VNScope into monitor into AV-IN; Press **ON-OFF** switch and wait for **10-15 secs** till image appears on the screen. Check the battery charge level;

3. Power OFF

Press **ON-OFF** switch for 3secs to switch off the monitor

4. Operation modes

Intubation mode(Record): perform intubation with connected VNScope To record film* press button once to open an information screen and press it again to start recording. To stop recording press to open information screen and press once more to stop recording.

Intubation mode(Photo): perform intubation with connected VNScope To take picture* press It takes 3 secs to finish processing.

5. Play back mode

*Recording will be performed only if TF memory card is inserted.

5.Play back Video/Photo

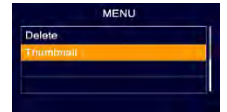
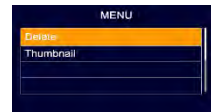
Select play back mode, press or to select Photo or Video ,according to file format, JPG or AVI .

To start play back video press to stop press again

6.Delete Video/Photo

Select play back mode select the picture, press ,select "Delete", press , select "Current" press to delete. To display thumbnail of films and pictures press and to choose "Thumbnail" mode press again and chose the file, press to delete

7.Zoom of image*

In intubation mode use to increase magnification

*In normal conditions no need to increase magnification as it decreases quality of image.

8. System settings

Keep "Loop Recording" –"OFF" to prevent scratch of previous records and "Screen Off" –"OFF" to avoid sudden switch off of the screen

Monitor warranty

This product's warranty, provided by our company, covers a period of 1 year from the date of purchase. All faulty parts and/or functions, resulting despite the user's normal use, will be repaired and/or replaced at no charge during the warranty period. However, if any of the malfunctions are caused by user carelessness, inadequate maintenance, or natural disaster, we will provide repair and/or replacement services for a fee regardless of the warranty period. In the event, our product does not prove suitable for your application you can return the product for an exchange or refund. To claim, the product must be returned in good condition within 14 days. Before returning a product, please contact and provide us a serial number. The customer is responsible for all shipping costs.

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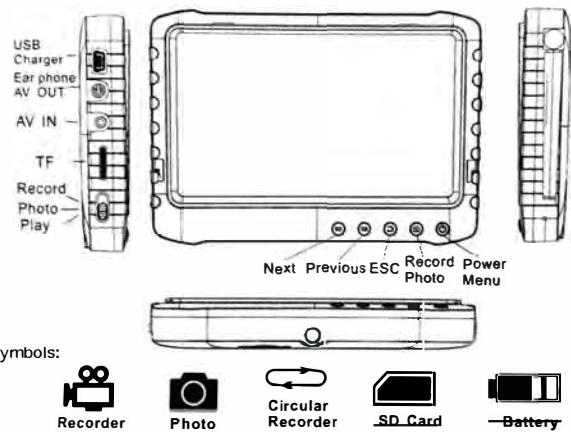
- Do not use or store the product in an environment with high temperature and/or high humidity to avoid damage. The Monitor/DVR is designed for use in temperatures between 0-40°C (32-104°F) and moderate humidity level (35 to 65% RH).
- Avoid direct sunlight..
- Do not remove or insert SD card when the power of the DVR is ON to avoid damaging files.
- Please do not dismantle the Monitor without authorization. This will violate warranty.
- Do not change or poke the battery. Do not put the battery into fire.
- Do not drop device.
- We recommend to backup the data in case of any loss caused by unstable power supply.
- When inserting SD card, make sure to use the right direction. Otherwise it might damage the card slot. The company is not responsible for any damage of that kind.
- Do not use the product to violate the privacy of others or any illegal activities. We shall not be responsible of illegal acts committed by the user.
- Do not use substances containing alcohol, benzene, thinners or other flammable substances to clean or maintain the device

Basic Information

- **Display:** 5-inch HD screen (800x480pix).
- **Recording:** Supports seamless loop video recording.
- **Photo:** Supports function of taking pictures from cameras.
- **AV IN:** TV-IN (NTSC/PAL).
- **AV OUT:** (NTSC/PAL)
- **Battery:** Built-in 3200mAh high capacity Li-battery, which provides a continuous record of over 10 hours.
- **Storage:** Supports up to 32GB TF card.
- **Language:** Supports multi-languages.
- **Supports Power-saving Mode:** Automatic shutdown & automatic off-screen.
- **Automatic upgrade:** Supports system's automatic upgrade.
- **Sunshade cover (optional):** Allows viewing screen under sunlight.
- **Screw-socket:** 1/4 inch standard screw socket.

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Please notice:

- a. The provided charger is for the DVR use only. Do not store it in a humid environment. Do not repair, break or make any changes to the adapter.
- b. For better performances, a new battery should be charged for more than 6 hours, but less than 10. It will reach its best performances after few times it will be charged and discharged.
- c. If the battery is not in use, it is recommended to charge the monitor every 2-3 months and then keep it after switch off.

Basic Operation

1. **Power On/Off:**
Press the "⏻" button for 3 sec to switch the DVR On. Hold on the "⏻" button again to switch it off.
2. **Working Mode switches** (located on the lower left corner):
 - "📹": DVR is on receiving/record/photo mode;
 - "▶": DVR is in play mode;
 - "📷": DVR is on photo mode.
3. **External storage card:**
The DVR will be able to record only if the storage card is inside. A symbol of the SD card will appear on the screen as well as its capacity.
4. **Working as a U-disk:**
Connect DVR to computer and switch the power on. Open the removable disk folder in the computer and find the media in the "RECORD" & "PHOTO" folders, under the "DCIM" directory.

Record & Photo

Switch DVR power on, and select the record/photo mode "📹".

1. **Record:** When on "📹" mode, press the "📷" button for 3 sec, and the DVR will start recording, showing the sign "REC". Press "📷" again to pause, the "||" sign will appear; press "📷" again to continue recording. Press "⏻" to stop, it will automatically save and exit. If you do not press "⏻", the system will automatically save the video into one segment, until the SD card will be full.

2. **Photo:** When on "📷" mode, press the "📷" button to take the picture.

Record & Photo

While using the "OVERWRITE" mode (press " " to go to MENU, select FUNCTION),

Do not press " " to end recording.

The system will automatically save video in segments.

When the card is full, it will overwrite the earliest recording, save the last one and will keep cycle recording.

3. Mute: Not applicable for this model.

Play Video/Photo

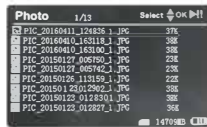
When the mode " " is on, the DVR will play video/photo. Press " " / " " to select "PHOTO" or "VIDEO". Press " " to exit and go back to main menu.

1. **Play Video:** After selecting "VIDEO", press " " to enter the video folder. Press " " / " " to select a date and press " " to enter the folder. Press " " / " " to select, and press " " to play video. Press a long " " to fast forward; Press a long " " to rewind; Press " " to pause; Press " " again to release pause; Press " " to exit.

Play Video/Photo

2. **View Photos:** select "PHOTO", press " " to enter the folder. Press " " / " " to select pictures and press " " to view the photo.. Press " " to view the next one; Press " " to view previous one Press " " to exit.

2. **Delete Photos & Videos:** Press " " / " " to select files. Press " " to open menu. Press " " / " " to select between "ALL/CURRENT" / "CANCEL". Press " " to confirm.

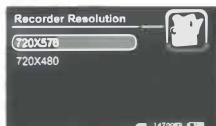
Function Settings

Switch DVR on. Select working mode " " and press " " to enter the "FUNCTION" and "SYSTEM" menu. Press " " / " " to select "FUNCTION", then press " " to enter to the menu.

1. **Channel Settings:** Not applicable for this model.
2. **Recorder Resolution:** Press " " / " " to select "RECORDER RESOLUTION", then press " " to enter. Press " " / " " to select "720x576" / "720x480". Press " " to confirm and exit.

Function Settings

3. **Frame Rate:** Press " " / " " to select "FRAME RATE", then press " " to enter. Press " " / " " to select "30 FPS" / "25 FPC", and press " " to confirm and exit.

4. **Video Segment Size:** Press " " / " " to select "VIDEO SEGMENT SIZE", then press " " to enter. Press " " / " " to select "10 MIN" / "30 MIN" / "60 MIN" and press " " to confirm and exit.

Function Settings

5. **Overwrite:** Press " " / " " to select "OVERWRITE", then press " " to enter. Press " " / " " to select "ON" / "OFF", and press " " to confirm and exit. When "Overwrite" is on, DVR will record in loop and save the latest video.

6. **Time Stamp:** Press " " / " " to select "TIME STAMP", then press " " to enter. Press " " / " " to select "ON" / "OFF", and press " " to confirm and exit. When "Time Stamp" is on, the date and time of recording will be recorded to video images.

Function Settings

7. **Information Display:** Press " " / " " to select "INFORMATION DISPLAY", then press " " to enter. Press " " / " " to select "ON" / "OFF", and press " " to confirm and exit. When "Information Display" is on, all the relevant info will appear on screen.

8. **Video Settings:** Press " " / " " to select "VIDEO SETTINGS", then press " " to enter. Press " " / " " to select "BRIGHTNESS" / "CONTRAST", and press " " to confirm and exit. Press " " / " " to adjust and press " " to exit.

System Settings

Switch DVR on. Select working mode " " and press " " to enter the "FUNCTION" and "SYSTEM" menu. Press " " / " " to select "SYSTEM", then press " " to enter to the menu.

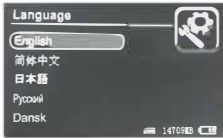
1. **Date Time:** Press " " / " " to select "DATE TIME", then press " " to enter. Press " " again to move the cursor, and press " " / " " to adjust Date & Time. Press " " to exit.

System Settings

2. **Language:** Press "▶▶"/"◀◀" to select "LANGUAGE", then press "⌂" to enter. Press "▶▶"/"◀◀" to choose the suitable Language. Press "⌂" to confirm and exit.

3. **AV Out:** Press "▶▶"/"◀◌" to select "AV OUT", then press "⌂" to enter. Press "▶▶"/"◀◌" to choose "PAL" / "NTSC". Press "⌂" to confirm the video displayed on the TV screen. Pull the AV cable out to resume watching through the DVR monitor.

System Settings

4. **Format:** Press "▶▶"/"◀◌" to select "FORMAT", then press "⌂" to enter. Press "▶▶"/"◀◌" to choose "YES" / "NO". Press "⌂" to confirm and exit.

5. **Power Save:** Press "▶▶"/"◀◌" to select "POWER SAVER", then press "⌂" to enter:

System Settings

1. **Auto LCD Off:** Press "▶▶"/"◀◌" to select "AUTO LCD OFF", then press "⌂" to enter. Press "▶▶"/"◀◌" to select "1 MIN" / "5 MIN" / "OFF". Press "⌂" to confirm and exit. When "Auto LCD Off" is On, the screen will automatically shut down once the set time is over. Press any key to resume the display.

System Settings

2. **Auto Power Off:** Press "▶▶"/"◀◌" to select "AUTO POWER OFF", press "⌂" to enter. Press "▶▶"/"◀◌" to select "5 MIN" / "10 MIN" / "OFF".

Press "⌂" to confirm and exit.

When "Auto Power Off" is On, the DVR will automatically shut down once the set time is over.

System Settings

6. **Firmware Update:** Press "▶▶"/"◀◌" to select "FIRMWARE UPDATE", then press "⌂" to enter. Press "▶▶"/"◀◌" to select "YES" / "NO". Press "⌂" to start update. Save the update file "zburnT_C11.run.elf" at the root directory of the SD card.

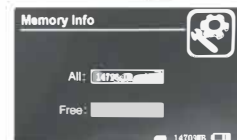
System Settings

7. **Default Setup:** Press "▶▶"/"◀◌" to select "DEFAULT SETUP", press "⌂" to enter. Press "▶▶"/"◀◌" to select "YES" / "No". Press "⌂" to confirm and exit. Press "⏪" to exit.

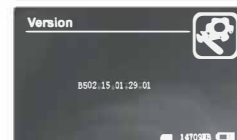
8. **Memory Info:** Press "▶▶"/"◀◌" to select "MEMORY INFO", and press "⌂" to enter. to the capacity of the SD card. Press "⏪" to exit.

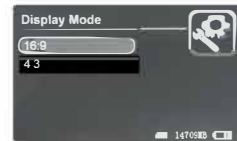
System Settings

9. **Version:** Press "▶▶"/"◀◌" to select "VERSION", and press "⌂" to view version information. Press "⏪" to exit.

10. **Display Mode:** Press "▶▶"/"◀◌" to select "DISPLAY MODE". Press "▶▶"/"◀◌" to choose "16:9" / "4:3". Press "⌂" to confirm. Press "⏪" to exit.

Troubleshooting

Troubleshooting:

- System does not turn on: Recharge the DVR. If the problem persists even after you have taken the above action, check the power supplier and replace it with a new one if necessary.
- System not responding: Press the RESET button in the pinhole (lower left side).
- Video can not be played: Check video format, to see if it is supported by the DVR system. Convert file if necessary. (MPEG-AVI)
- Unreadable text: Make sure that the LANGUAGE function is on the correct language.
- System can't upload/download files or is not recognized by PC: Replace USB cable or port.
- File cannot be saved: Make sure the SD card is inserted and that it is not full. Delete files to make space or FORMAT to fix damaged areas.

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User Manual

Warnings

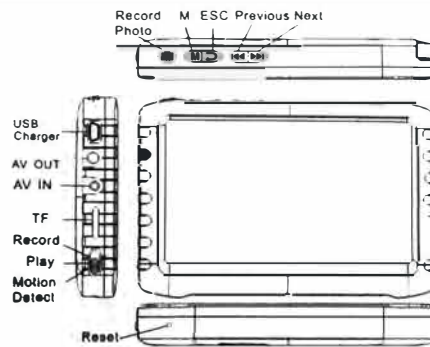
Please read the following instructions carefully before using the device:

- Do not use or store the product in an environment with high temperature and/or high humidity to avoid damage. The Monitor/DVR is designed for use in temperatures between 0-40°C (32-104°F) and moderate humidity level (35 to 65% RH).
- Avoid direct sunlight.
- Do not remove or insert SD card when the power of the DVR is ON to avoid damaging files.
- Please do not dismantle the Monitor without authorization. This will violate warranty.
- Do not change or poke the battery. Do not put the battery into fire.
- Do not drop device.
- We recommend to backup the data in case of any loss caused by unstable power supply.
- When inserting SD card, make sure to use the right direction. Otherwise it might damage the card slot. The company is not responsible for any damage of that kind.
- Do not use the product to violate the privacy of others or any illegal activities. We shall not be responsible of illegal acts committed by the user.
- Do not use substances containing alcohol, benzene, thinners or other flammable substances to clean or maintain the device

Basic Information

- **Display:** 5-inch HD screen (800x480pix).
- **Recording:** Supports seamless loop video recording.
- **Photo:** Supports function of taking pictures from cameras.
- **AV IN:** TV-IN (NTSC/PAL).
- **AV OUT:** (NTSC/PAL)
- **Battery:** Built-in 3200mAh high capacity Li-battery, which record continuously over 10 hours.
- **Storage:** Supports up to 32GB TF card.
- **Language:** Supports multi-languages.
- **Supports Power-saving Mode:** Automatic shutdown & automatic off-screen.
- **Automatic upgrade:** Supports system's automatic upgrade.

Buttons & Interface

Symbols:

How to Charge

1. Battery capacity symbols:

Note: a running out battery might cause the monitor not to switch on. Make sure to charge it by connecting to an AC adapter or PC.

2. The battery needs to be charged 4-6 hours for full functioning. Unplug the charger once the battery status shows full.

Please notice:

- The provided charger is for the DVR use only. Do not store it in a humid environment. Do not repair, break or make any changes to the adapter.
- For better performances, a new battery should be charged for more than 6 hours, but less than 10. It will reach its best performances after few times it will be charged and discharged.
- If the battery is not in use, it is recommended to charge the monitor every 2-3 months and then keep it after switch off.

Basic Operation

- Power On/Off:**
Press the "M" button for 3 sec to switch the DVR On. Hold on the "M" button again to switch it off.
- Working Mode switches** (located on the lower left corner):
 - "Video camera": DVR is on receiving/record/photo mode;
 - "Play": DVR is in play mode;
 - "Hand holding camera": DVR is on motion detect mode.
- External storage card:**
The DVR will be able to record only if the storage card is inside. A symbol of the SD card will appear on the screen as well as its capacity.
- Working as a U-disk:**
Connect DVR to computer and switch the power on. Open the removable disk folder in the computer and find the media in the "RECORD" & "PHOTO" folders, under the "DCIM" directory.

Record & Photo

Switch DVR power on, and select the record/photo mode "Video camera".

- Photo:** When on "Video camera" mode, press the "Photo" button once, the symbol "Photo" will appear, and you can take a picture.

- Record:** When on "Video camera" mode, press the "Record" button for 3 sec, and the DVR will start recording, showing the sign "REC". Press "Photo" again to pause, the "||" sign will appear; press "Photo" again to continue recording. Press "Play" to stop, it will automatically save and exit. If you do not press "Photo", the system will automatically save the video into one segment, until the SD card will be full.

Record & Photo

While using the "OVERWRITE" mode (press "↶" to go to MENU, select FUNCTION), Do not press "⏻" to end recording. The system will automatically save video in segments. When the card is full, it will overwrite the earliest recording, save the last one and will keep cycle recording.

3. Channel Switch: Not applicable for this model.

4. Mute: Not applicable for this model.

Motion Detect

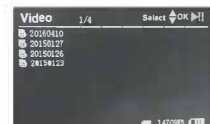
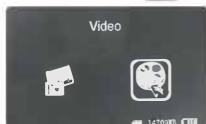
- Switch DVR on, and select the motion detect mode : "👤". To exit and go to system/function menu, press "↶".
- When on "👤" mode, press "⏻" for 3 sec to enter the motion detect recording. Once a moving object will appear, the system will recognize it and start automatically to record it. A "REC" sign will be displayed on the screen.
If the system does not detect movement for 10 sec, it will stop the recording automatically, save the video, and will wait for the next motion to appear.
to stop the recording manually, press "⏻". It will save automatically and exit.
- If the "⏻" will not be pressed, the system will automatically save the video in segments until the storage card will be full. If the "OVERWRITE" mode is on, do not press "⏻" to end recording, the system will automatically save in segments. Once the card is full, it will overwrite the the earliest video and save the last one, and keep recording in loop.

Play Video/Photo

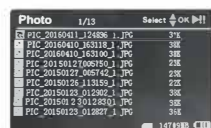
When the mode "▶" is on, the DVR will play video/photo. Press "⏪"/"⏩" to select "PHOTO" or "VIDEO". Press "↶" to exit and go back to main menu.

- Play Video:** After selecting "VIDEO", press "⏻" to enter the video folder. Press "⏪"/"⏩" to select a date and press "⏻" to enter the folder. Press "⏪"/"⏩" to select, and press "⏻" to play video. Press a long "⏩" to fast forward; Press a long "⏪" to rewind; Press "⏻" to pause; Press "⏻" again to release pause; Press "↶" to exit.

Play Video/Photo

- View Photo:** select "PHOTO", press "⏻" to enter the folder. Press "⏪"/"⏩" to select pictures and press "⏻" to view the photo.. Press "⏩" to view the next one; Press "⏪" to view previous one Press "↶" to exit.

- Delete Photo & Video:** Press "⏪"/"⏩" to select files. Press "M" to open menu. Press "⏪"/"⏩" to select between "ALL/CURRENT" / "CANCEL". Press "⏻" to confirm.

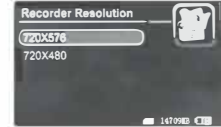
Function Settings

Switch DVR on. Select working mode "👤" and press "↶" to enter the "FUNCTION" and "SYSTEM" menu. Press "⏩"/"⏪" to select "FUNCTION", then press "⏻" to enter to the menu.

- Channel Settings:** Not applicable for this model.

- Recorder Resolution:** Press "⏩"/"⏪" to select "RECORDER RESOLUTION", then press "⏻" to enter. Press "⏩"/"⏪" to select "720x576" / "720x480". Press "⏻" to confirm and exit.

Function Settings

- Frame Rate:** Press "⏩"/"⏪" to select "FRAME RATE", then press "⏻" to enter. Press "⏩"/"⏪" to select "30 FPS" / "25 FPC", and press "⏻" to confirm and exit.

- Video Segment Size:** Press "⏩"/"⏪" to select "VIDEO SEGMENT SIZE", then press "⏻" to enter. Press "⏩"/"⏪" to select "10 MIN" / "30 MIN" / "60 MIN" and press "⏻" to confirm and exit.

Function Settings

- Overwrite:** Press "⏩"/"⏪" to select "OVERWRITE", then press "⏻" to enter. Press "⏩"/"⏪" to select "ON" / "OFF", and press "⏻" to confirm and exit. When "Overwrite" is on, DVR will record in loop and save the latest video.

- Time Stamp:** Press "⏩"/"⏪" to select "TIME STAMP", then press "⏻" to enter. Press "⏩"/"⏪" to select "ON" / "OFF", and press "⏻" to confirm and exit. When "Time Stamp" is on, the date and time of recording will be recorded to video images.

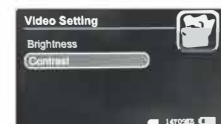
Function Settings

- Information Display:** Press "⏩"/"⏪" to select "INFORMATION DISPLAY", then press "⏻" to enter. Press "⏩"/"⏪" to select "ON" / "OFF", and press "⏻" to confirm and exit. When "Information Display" is on, all the relevant info will appear on screen.

- Video Settings:** Press "⏩"/"⏪" to select "VIDEO SETTINGS", then press "⏻" to enter. Press "⏩"/"⏪" to select "BRIGHTNESS" / "CONTRAST", and press "⏻" to confirm and exit. Press "⏩"/"⏪" to adjust and press "↶" to exit.

System Settings

Switch DVR on. Select working mode / and press " / "" to enter the "FUNCTION" and "SYSTEM" menu. Press " / "" to select "SYSTEM", then press "" to enter to the menu.

1. **Date Time:** Press " / "" to select "DATE TIME", then press "" to enter. Press "" again to move the cursor, and press " / "" to adjust Date & Time. Press "" to exit.

System Settings

2. **Language:** Press " / "" to select "LANGUAGE", then press "" to enter. Press " / "" to choose the suitable Language. Press "" to confirm and exit.

3. **AV Out:** Press " / "" to select "AV OUT", then press "" to enter. Press " / "" to choose "PAL" / "NTSC". Press "" to confirm the video displayed on the TV screen. Pull the AV cable out to resume watching through the DVR monitor.

System Settings

4. **Format:** Press " / "" to select "FORMAT", then press "" to enter. Press " / "" to choose "YES" / "NO". Press "" to confirm and exit.

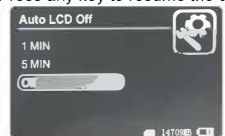
5. **Power Save:** Press " / "" to select "POWER SAVER", then press "" to enter:

System Settings

1. **Auto LCD Off:** Press " / "" to select "AUTO LCD OFF", then press "" to enter. Press " / "" to select "1 MIN" / "5 MIN" / "OFF". Press "" to confirm and exit. When "Auto LCD Off" is On, the screen will automatically shut down once the set time is over. Press any key to resume the display.

2. **Auto Power Off:** Press " / "" to select "AUTO POWER OFF", press "" to enter. Press " / "" to select "5 MIN" / "10 MIN" / "OFF". Press "" to confirm and exit. When "Auto Power Off" is On, the DVR will automatically shut down once the set time is over.

System Settings

6. **Firmware Update:** Press " / "" to select "FIRMWARE UPDATE", then press "" to enter. Press " / "" to select "YES" / "No". Press "" to start update. Save the update file "zburnT_C11_run.elf" at the root directory of the SD card.

7. **Default Setup:** Press " / "" to select "DEFAULT SETUP", press "" to enter. Press " / "" to select "YES" / "No". Press "" to confirm and exit. Press "" to exit.

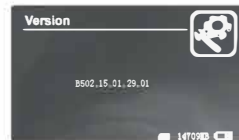
System Settings

8. **Memory Info:** Press " / "" to select "MEMORY INFO", and press "" to enter. to the capacity of the SD card. Press "" to exit.

9. **Version:** Press " / "" to select "VERSION", and press "" to view version information. Press "" to exit.

System Settings

10. **Display Mode:** Press " / "" to select "DISPLAY MODE". Press " / "" to choose "16:9" / "4:3". Press "" to confirm. Press "" to exit.

Troubleshooting

Troubleshooting:

- ♦ System does not turn on: Recharge the DVR. If the problem persists even after you have taken the above action, check the power supplier and replace it with a new one if necessary.
- ♦ System not responding: Press the **RESET** button in the pinhole (lower left side).
- ♦ Video can not be played: Check video format, to see if it is supported by the DVR system. Convert file if necessary. (MPEG-AVI)
- ♦ Unreadable text: Make sure that the LANGUAGE function is on the correct language.
- ♦ System can't upload/download files or is not recognized by PC: Replace USB cable or port.
- ♦ File cannot be saved: Make sure the SD card is inserted and that it is not full. Delete files to make space or **FORMAT** to fix damaged areas.

Specifications

AV IN	TV system	NTSC / PAL
	Output voltage	3.6-4.2 V
AV OUT	TV system	NTSC / PAL
HD 5-inch LCD screen	Active area	108mm(W) X 64.8mm(H)
	Resolution	800 X 480 Pixel
	Luminance for LCM	200 cd/m ²
Video	Resolution	720*576 / 720*480
	Video compress	MPEG-AVI 30fps
	Play format	MPEG-AVI

Specifications

Photo	Format	JPG, JPEG
Storage	External SD card	TF card 1GB-32GB (optional)
	32GB recording time	10-32 hours
	1GB photos	15625pcs
Operating system	PC	Windows & Mac OS
	Port	USB 2.0
	Language	Multiple
Power	Voltage	DC 5V
	Current(receiving mode)	660MA
	Current(recording mode)	680MA
	Current (recording with offTscreen mode)	340MA
	AC adapter	DC 5V 2A
Standby time	Built-in battery record	Li-battery 3200Mah ≥12 hours(off-screen); ≥6hours (on-screen) *LED camera not included.
Size (L x W x H)		131mm x 86mm x 18mm
Weight(g)		205g

Package Content

The following items are in the package:

DVR monitor	1 unit
User manual	1 unit
Usb cable	1 unit
AV-IN cable	1 unit
AV-OUT cable	1 unit
AC adapter	1 unit
Back holder	1 unit

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Evgeny Pecherer
Chief Executive Officer
Peak Medical Ltd.

EMC requirements

IEC 60601 -1-2 Edition 4.0 (2014)

Environment of intended uses:

Professional Healthcare Facility Environment

Summary of Test Results

Test	Standard	Class/ Severity level	Test result
Emission (IEC 60601 -1-2 section 7.1)			
Radiated emission Freq. range: 30 - 1000 MHz	CISPR 11	Group 1 Class A	Complies
Immunity (IEC 60601 -1-2 sections 8.9, 8.10 & 8.11)			
Immunity from Electrostatic discharge (ESD)	IEC 61000 -4-2	8 kV contact discharges & 15 kV air discharges	Complies
Immunity from radiated electromagnetic fields	IEC 61000 -4-3	3.0 V/m; 80 MHz ÷ 2.7 GHz, 80% AM, 1 kHz	Complies
Immunity from Proximity field from wireless communications equipment	IEC 61000-4-3	List of frequencies, from 9 V/m up to 28 V/m, PM (18 Hz or 217 Hz), FM 1 kHz	Complies
Immunity from conducted disturbances induced by radio-frequency fields	IEC 61000 -4-6	3.0 & 6.0 VRMS on Video cable; 0.15÷ 80 MHz, 80% AM 1 kHz	Complies
Immunity from power frequency magnetic field	IEC 61000 -4-8	30 A/m @ 50 Hz & 60 Hz	Complies
Immunity from proximity frequency magnetic field	Section 8.11	65 A/m @ 134.2 kHz; PM 2.1kHz 50%; 7.5 A/m @ 13.56 MHz; PM 50kHz 50%	Complies

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© 2020 KREIENBAUM Neoscience GmbH. Alle Rechte vorbehalten. Eine Reproduktion dieses Dokumentes in jeglicher Form ist nur mit vorheriger schriftlicher Genehmigung der KREIENBAUM Neoscience GmbH erlaubt. Andere in diesem Dokument möglicherweise verwendete Warenzeichen und Handelsbezeichnungen sind unter Umständen Marken und Namen der entsprechenden Firmen oder ihrer Produkte.